

Dentobuff® Strip Estimation of salivary buffering capacity



Cat. No. 67670



Sample collection



Sample application



Interpretation at 5 minutes

Dental tests



Dentocult® SM Strip mutans Detection of mutans streptococci in dental plaque and stimulated saliva



Cat. No. 67647



Sample collection using Screening Strips



Sample application using Site Strips



Attach the strips to the cap and recap the vial



Incubation (48 hours)



Interpretation (Screening Strips)



Interpretation (Site Strips)

Dentocult® LB Detection of lactobacilli in stimulated saliva



Cat. No. 67671



Sample collection



Sample application



Incubation (4 days)



Interpretation

Dentocult® CA Detection of yeasts (especially Candida) in stimulated saliva and soft-tissue lesions



Cat. No. 68291



Sample application



Incubation (48 hours)



Interpretation

DENTAL

Cultura M allows you to perform the test in the office.

Cultura M
small, durable incubator suitable for office use

Cat. No. 05866



Dental tests

Dentobuff® Strip



Buffering capacity



low



medium



high

Dentocult® SM Strip mutans



Colony density



0



1



2



3



Site Strip

Dentocult® LB



Colony density, CFU/ml



10^2



10^4



10^5



10^6

Dentocult® CA



Colony density, CFU/ml



10^3



10^4



10^5



10^6



Orion Diagnostica Oy
P.O.Box 83
FI-02101 Espoo, Finland
Tel. +358 10 4261
Fax +358 10 426 2794
www.oriondiagnostica.com